


INSTRUCTIONS FOR WEARING AND CARING FOR YOUR MASK

Always wash your hands before and after putting on or taking off the mask and once it's on, it shouldn't be touched. The folds face downward so the nothing gets caught in them and kept close to your face.

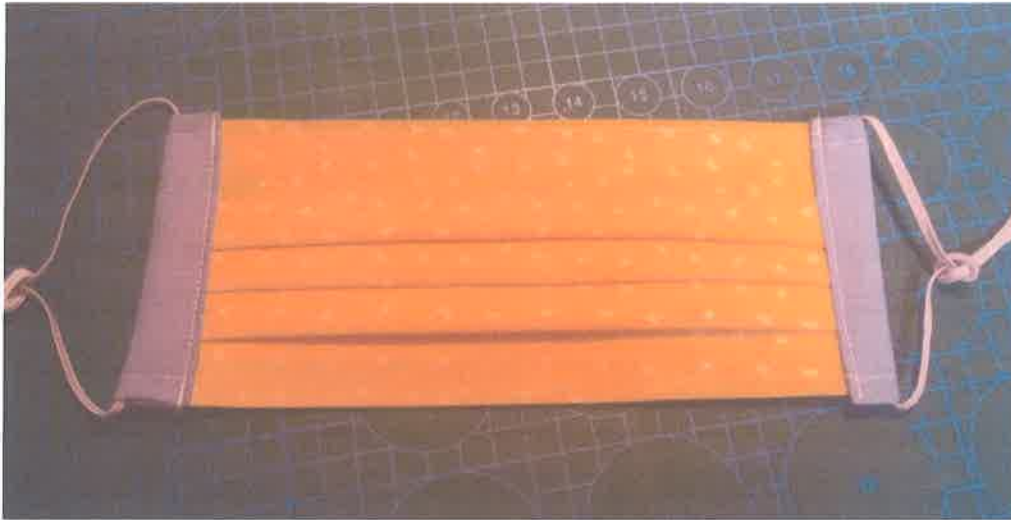

The elastics are loosely tied in a knot so that you can custom fit the mask to your face. The casings (in this case the blue) should be pulled tight so that it forms a close fit along your cheeks.

On the back (in all cases it's a beige fabric), there is folded over portion at the bottom. This is a filter pocket. If you would like an added layer of filtering medium, you can insert it there.



At the top of the back is a place for a nose pinch. This will keep the mask close around your nose. You can use different materials such as pipe cleaners or twist ties. These will break eventually, so will need replacing at some point. Pipe cleaners are included so you can try them.

The tag at the bottom is required but can be tucked inside the backing piece so it won't be against your skin.



LAUNDERING INSTRUCTIONS

As for laundering, they should be washed in warm water and laid flat to dry. When in the wash they should be put in a lingerie bag or pillowcase so the elastics don't get tangled in other laundry. If they are going to be ironed, just be careful of the elastic, it has been known to melt if left on the heat too long.